










## Exam Glove Quick Reference Guide

	Patient Care	Oxvir (Accelerated hydrogen peroxide)* Maximum Duration	Hazardous Drugs
<b>Sensicare ICE Nitrile (250 ea)</b> 	✓	✓ 20 minutes	✓ Except Carmustine and Thiopeta
<b>Sensicare Nitrile (150 ea)</b> 	✓	✓ 20 minutes	✓ Except Carmustine and Thiopeta
<b>Aurelia Sonic Nitrile</b> 	✓	✓ 5 minutes	✓ Except Carmustine and Thiopeta
<b>Synguard Nitrile</b> 	✓	✗	✓ See <a href="#">PCN</a> for additional info
<b>Restore Nitrile</b> 	✓	✗	✓ Except Carmustine and Thiopeta
<b>Sensicare Silk Nitrile</b> 	✓	✓ 10 minutes	✓ See <a href="#">PCN</a> for additional info
<b>Vinyl Exam</b> 	✓	✗	✗
<b>VersaShield Extended Cuff**</b> 	✓	✓ 35 minutes	✓
<b>Curad Extended Cuff**</b> 	✓	✓ 10 minutes	✓

\*Gloves are not required for Oxvir (accelerated hydrogen peroxide) use less than 5 minutes. Hand rinsing is recommended.

\*\*Extended cuff gloves should be reserved for tasks specifically identified in the [Non Sterile Exam Glove Selection Guide](#).

Refer to the [Product Change Notice](#) or [Non Sterile Exam Glove Selection Guide](#) for more detailed information and wear times with specific chemicals or hazardous drugs.